

CLC CONIC™

IMPORTING CAD LIBRARIES INTO DENTAL WINGS' DENTAL SYSTEM




passion for dental excellence


STEP-BY-STEP INSTRUCTIONS

Importing CAD libraries into Dental Wings' Dental System


1. Open DWClient

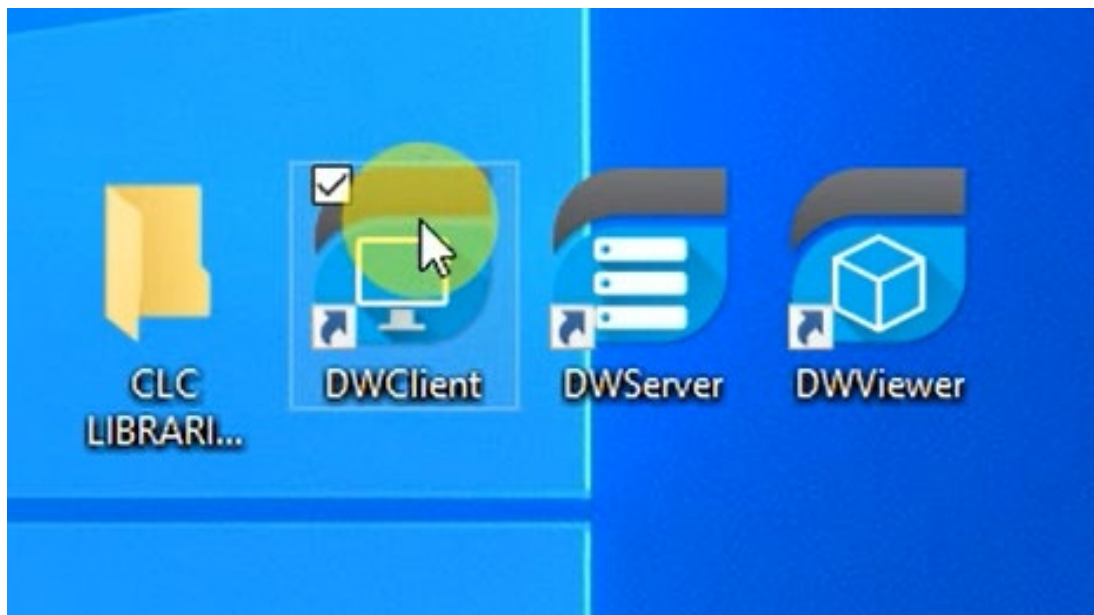


2. Choose «Settings»

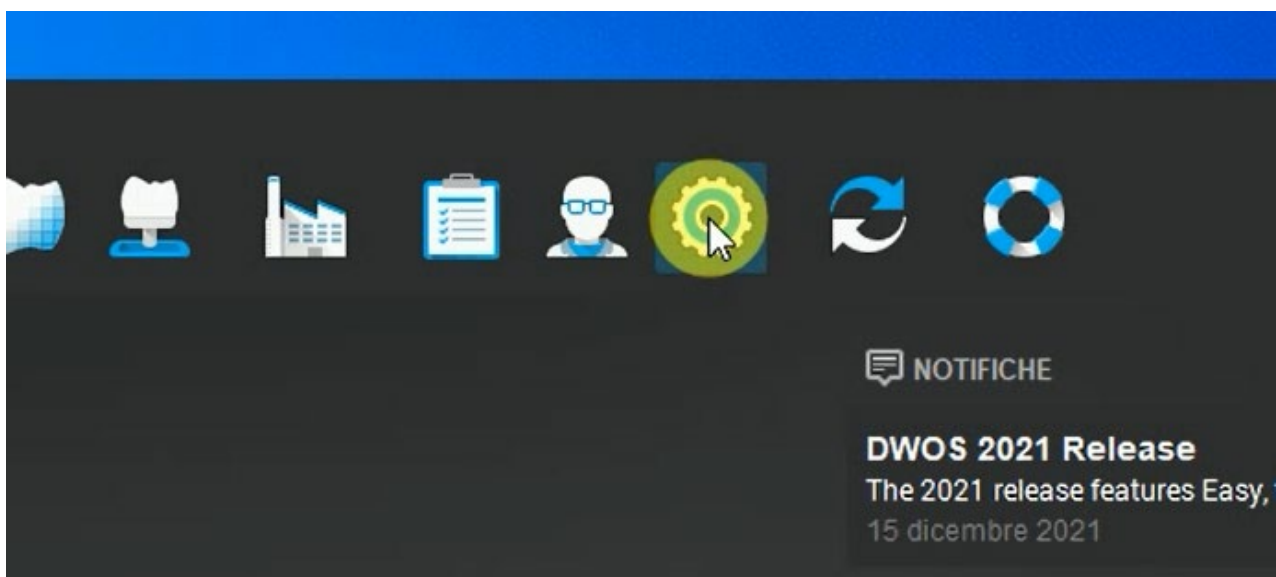


3. Choose «Implant kit»

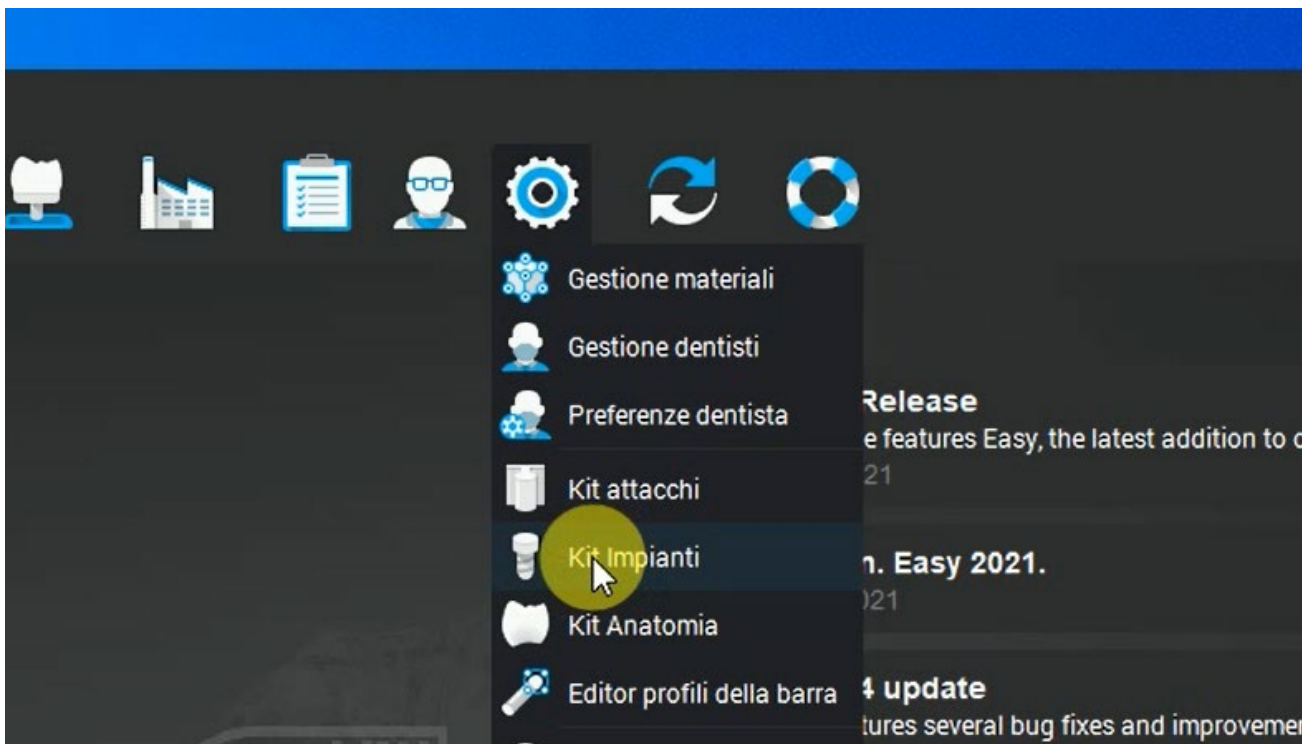


4. Choose Menu

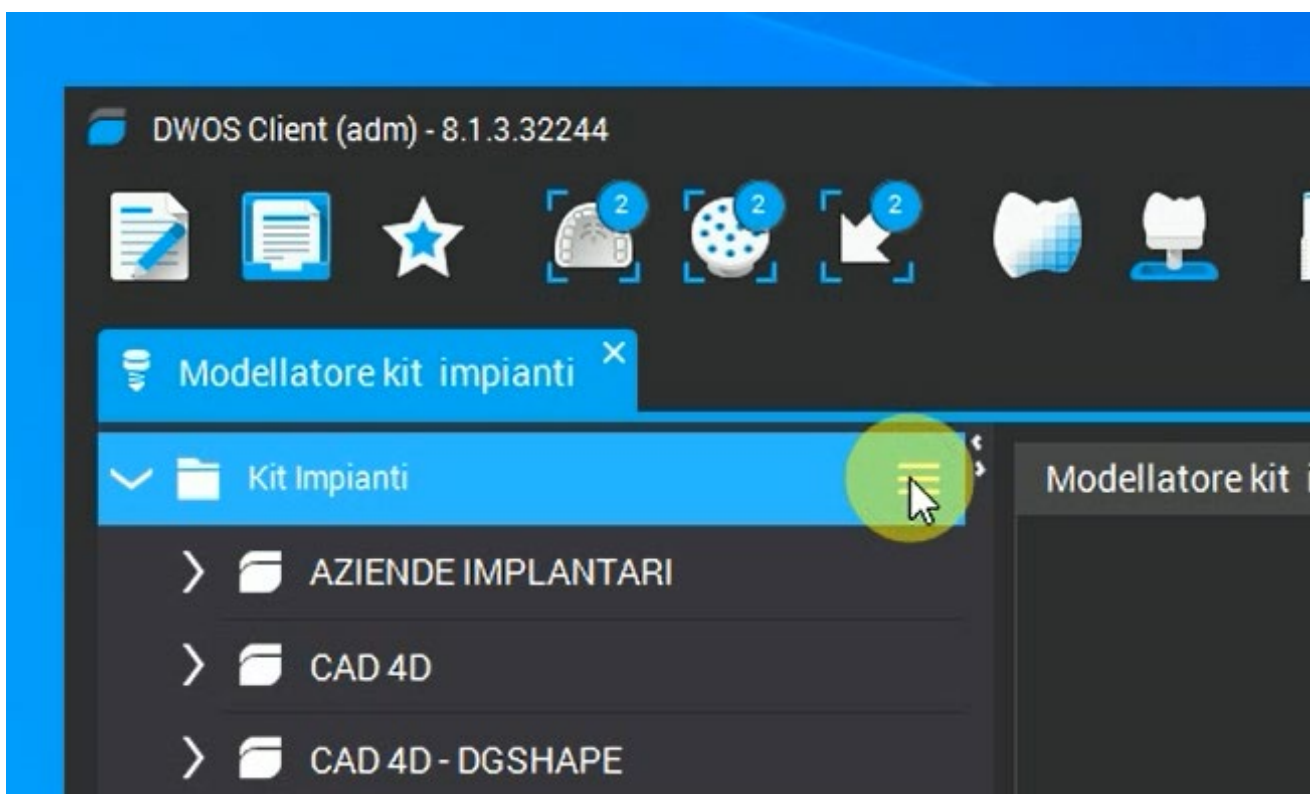

5. Choose «Import implant manufacturer kit»

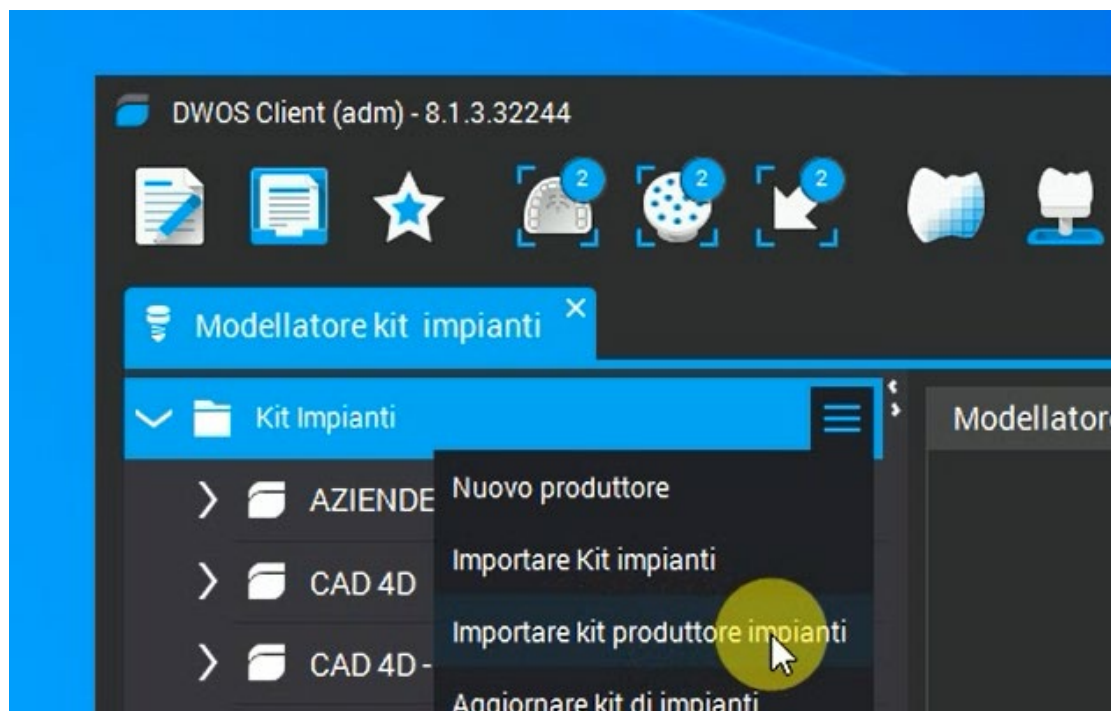

6. Select CLC digital library for Dental Wings and clic to «Open it»

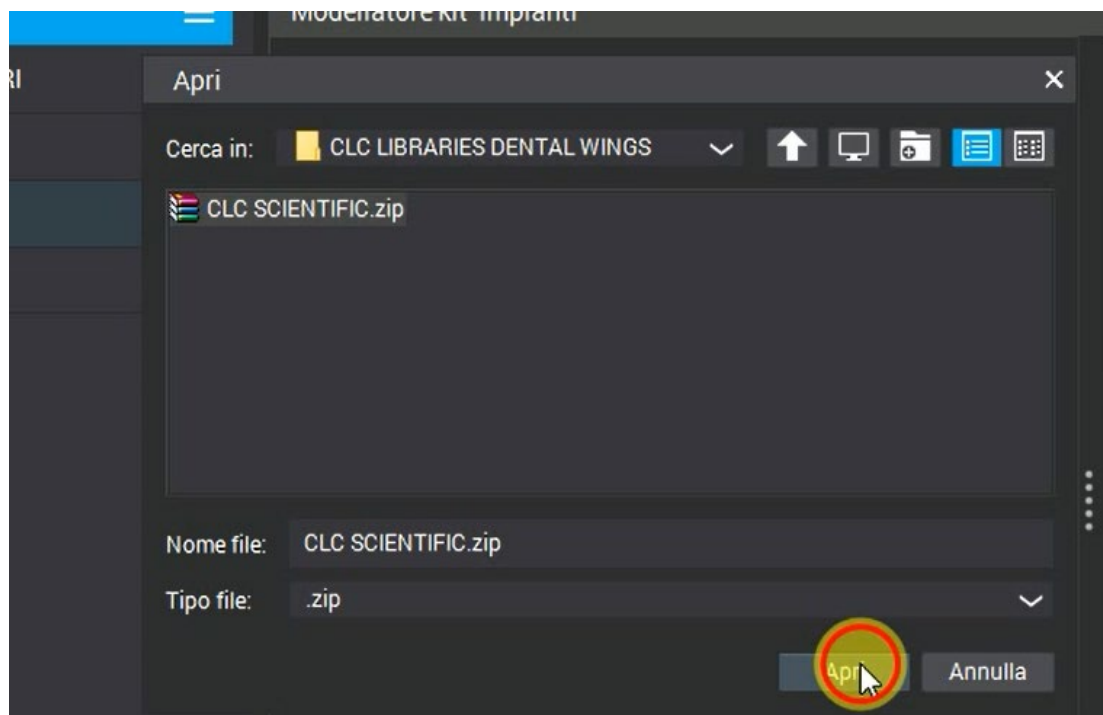

7. Wait for the import process to complete and then use the libraries for your projects





passion for dental excellence

Contrà Pasini 18, 36100 Vicenza - Italy
Tel. +39 0444 291168

www.clcscientific.com
administration@clcscientific.com